

The following are our tips for cleaning your cattle repeater syringes

1. The sooner you clean after using, the better
2. No soap or disinfectant, residue can affect efficacy of modified live vaccines
3. Clean the inside by drawing hot water (<180 degrees) and squirting it out 3-5 times
4. Use a bottle brush to clean out any left over vaccine residue
5. Dry thoroughly and apply lubricant to O ring before reassembling
6. If your gun is hard to plunge you can add glycerine or food grade lubricant to the O ring
7. Change your needle frequently, always use a clean needle to puncture a bottle